

## Maintenance & Safety Inspections

To ensure optimum performance the Propulse III Universal Electronic Ear Irrigator should be serviced every 12 months. Service or repairs conducted by unauthorised agencies/organisations invalidate any or implied warranties from Mirage.

The Propulse III Universal should under-go routine electrical safety testing to ensure that it remains safe to use.

Users of the Propulse III Universal should carry out regular inspections to ensure that the handle and hose, power adaptor and cable, reservoir and main body of the unit are free from damage prior to use. If any damage is evident the Propulse III Universal should **NOT** be used until replacement parts have been fitted. Only Propulse branded items should be used with the Propulse III.

The Propulse III Universal is not user repairable and should be returned to your Propulse supplier or Mirage Service Department (UK Only) for service/and or repair:

### **Mirage Service Department**

53 Purbeck Drive, Lostock, Bolton,  
Lancashire, BL6 4JF

Tel: 0845 130 5445